

WWW.BIOTEKE.CN

IVD CE



SARS-CoV-2 Antigen Test Kit



A

Simple operation, easy to read results. Get positive results as fast as 20 seconds.



High sensitivity and specificity. Low possibility of misdiagnosis.



FDA CE compliance certification. Safe & reliable.



INTENDED USE & PRINCIPLE



The kit is only used for in vitro qualitative detection of SARR-CoV-2 anitgen from human anterior nasal swab specimens.

Principle

The SARS-CoV-2 Antigen test kit is based on immunochromatographic and use double-antibody sandwich method to detect SARS-CoV-2 antigen.When the concentration of COVID-19 antigen in specimen is higher than the minimum detection limit,the viral antigen will form complexes with labeled antibodies first.Under chromatography,the complexes move forward along the nitrocellulose membrane till captured by pre-coated monoclonal antibody of COVID-19 in detection zone.Through the visible color to interprete the test results.

BioTeke

Anterior Nasal Swab

SARS-CoV-2 Antigen Test Kit

INTENDED PURPOSE: This kit is only used for the *in vitro* qualitativ detection of SARS-CoV-2 antigen from human Safe

E/1

place tube here



Nucleic acid PCR detection method is till the gold standard for SARS-CoV-2 detection.

The pathogen can be detected in the early stages of infection, but the long TAT and cumbersome operation are its shortage. Antibody test is with short TAT and simple operation, but it has a window period. Generally, it can only be detected in 1-2 weeks after infection.





PACKING SPECIFICATION

Name	Components	Loading Quantity		
SARS-CoV-2 Antigen Rapid Test Kit		1 test/kit	3 tests/kit	5 tests/kit
	Test card	1pc	3pcs	5pcs
	Sample extraction tube	1pc	3pcs	5pcs
	Anterior nasal swab	1pc	3pcs	5pcs
	Swab	1pc	3pcs	5pcs

User manual, waste bag, QC included





+



S-CoV-2 Antigen Test Kit



RESULTS INTERPRETATION



POSITIVE

The C-line and T-line areas are both red/purple/pink bands



POSITIVE

The C-line and T-line areas are both red/purple/pink bands



POSITIVE

The C-line and T-line areas are both red/purple/pink bands



NEGATIVE

Only the C line area has red/purple/pink bands



INVALID

Red/purple/pink bands only in the T-line area, or no bands at all



INVALID

Red/purple/pink bands only in the T-line area, or no bands at all

















COVID,Flu A&B, HPIV,RSV,ADV,MP

7-in-1 ag test

COVID,Flu A&B, HPIV,RSV,ADV,MP, Strep A 8-in-1 ag test COVID,Flu A&B, HPIV2,HPIV1+3, RSV,ADV,MP 8-in-1 ag test







BioTeke IVD SUPPLIER



Rapid Test Kit

Fresh and simple linestyle enterprise product publicity



Molecular PCR Panel

Fresh and simple linestyle enterprise product publicity



Labs Equipments

Fresh and simple linestyle enterprise product publicity



OEM/ODM Service

Fresh and simple linestyle enterprise product publicity



Technical Support

Fresh and simple linestyle enterprise product publicity